



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	FUNCTION
<b>IMPLANT PREP</b>	
IM1S	special*
IM1 AL	implant
IM1 SP	implant
IM2A	implant
IM2A-15	implant
IM2P	implant
IM2P-15	implant
IM2.8A	implant
IM2.8P	implant
IM3A	implant
IM3A -15	implant
IM3P	implant
IM3P-15	implant
IM3.4A	implant
IM3.4P	implant
IM4A	implant
IM4P	implant
P2-3	implant
P3-4	implant
<b>OSTEOTOMY</b>	
OT1	cancellous / cortical
OT1A	cancellous / cortical
OT2	cancellous / cortical
OT3	cancellous / cortical
OT4	cancellous / cortical
OT5	cancellous / cortical
OT5A	cancellous / cortical
OT5B	cancellous / cortical
OT6	cancellous / cortical
OT7	cancellous / cortical
OT7A	cancellous / cortical
OT7S-3	special*
OT7S-4	special*
OT7-20	cancellous / cortical
OT8L	cancellous / cortical
OT8R	cancellous / cortical
OT9	cancellous / cortical
OT11	cancellous / cortical
OT12	cancellous / cortical
OT12S	special*
OT13	cancellous / cortical
OT14	cancellous / cortical
SLO-H	cancellous / cortical
PL1	cancellous / cortical
PL2	cancellous / cortical
PL3	cancellous / cortical
<b>OSTEOPLASTY</b>	
OP1	cancellous / cortical
OP2	cancellous / cortical
OP3	cancellous / cortical
OP3A	cancellous / cortical
OP4	cancellous / cortical
OP5	perio
OP5A	perio / special
OP6	perio

INSERT	FUNCTION
<b>OSTEOPLASTY</b>	
OP6A	perio
OP7	cancellous / cortical
OP8	perio / special
OP9	cancellous / cortical
SLC	cancellous / cortical
<b>MINI IMPLANT PREP</b>	
MDI 1.9	implant
MDI 2.2	implant
MDI 2.5	implant
<b>SINUS MEMBRANE ELEVATION</b>	
EL1	endo*
EL2	endo*
EL3	endo*
SLS	special*
SLE1	special*
SLE2	special*
<b>PERIOSTEUM</b>	
PR1	cancellous / cortical
PR2	cancellous / cortical
<b>ENDO RETRO</b>	
EN1	endo*
EN2	endo*
EN3	endo*
EN4	endo*
EN5R	endo*
EN5L	endo*
EN6R	endo*
EN6L	endo*
<b>EXTRACTION</b>	
EX1	cancellous / cortical
EX2	cancellous / cortical
EX3	cancellous / cortical
<b>EXPLANTATION</b>	
EXP3-R	special / cancellous / cortical
EXP3-L	special / cancellous / cortical
EXP4-R	special / cancellous / cortical
EXP4-L	special / cancellous / cortical
<b>PERIO</b>	
PP1	perio
PS1	perio
PS2	perio
PS6	perio
PP10	perio
PP11	perio
PP12	perio
<b>IMPLANT CLEANING</b>	
ICP + IC1	endo / perio
<b>RESTORATIVE</b>	
DB2	perio / special

\* Massima potenza consentita  
\* Maximum power permitted

\* Maximal zugelassene Leistung  
\* Puissance maximum autorisée

\* Máxima potencia permitida  
\* Maximalt tillåten effekt



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	MODE	POWER
--------	------	-------

### IMPLANT PREP

IM1S	BONE	SPECIAL*
IM1 AL	IMPL	IMPLANT
IM1 SP	IMPL	IMPLANT
IM2A	IMPL	IMPLANT
IM2A-15	IMPL	IMPLANT
IM2P	IMPL	IMPLANT
IM2P-15	IMPL	IMPLANT
IM2.8A	IMPL	IMPLANT
IM2.8P	IMPL	IMPLANT
IM3A	IMPL	IMPLANT
IM3A -15	IMPL	IMPLANT
IM3P	IMPL	IMPLANT
IM3P-15	IMPL	IMPLANT
IM3.4A	IMPL	IMPLANT
IM3.4P	IMPL	IMPLANT
IM4A	IMPL	IMPLANT
IM4P	IMPL	IMPLANT
P2-3	IMPL	IMPLANT
P3-4	IMPL	IMPLANT

### OSTEOTOMY

OT1	BONE	SPONGIOUS / CORTICAL
OT1A	BONE	SPONGIOUS / CORTICAL
OT2	BONE	SPONGIOUS / CORTICAL
OT3	BONE	SPONGIOUS / CORTICAL
OT4	BONE	SPONGIOUS / CORTICAL
OT5	BONE	SPONGIOUS / CORTICAL
OT5A	BONE	SPONGIOUS / CORTICAL
OT5B	BONE	SPONGIOUS / CORTICAL
OT6	BONE	SPONGIOUS / CORTICAL
OT7	BONE	SPONGIOUS / CORTICAL
OT7A	BONE	SPONGIOUS / CORTICAL
OT7S-3	BONE	SPECIAL*
OT7S-4	BONE	SPECIAL*
OT7-20	BONE	SPONGIOUS / CORTICAL
OT8L	BONE	SPONGIOUS / CORTICAL
OT8R	BONE	SPONGIOUS / CORTICAL
OT9	BONE	SPONGIOUS / CORTICAL
OT11	BONE	SPONGIOUS / CORTICAL
OT12	BONE	SPONGIOUS / CORTICAL
OT12S	BONE	SPECIAL*
OT13	BONE	SPONGIOUS / CORTICAL
OT14	BONE	SPONGIOUS / CORTICAL
SLO-H	BONE	SPONGIOUS / CORTICAL
PL1	BONE	SPONGIOUS / CORTICAL
PL2	BONE	SPONGIOUS / CORTICAL
PL3	BONE	SPONGIOUS / CORTICAL

### OSTEOPLASTY

OP1	BONE	SPONGIOUS / CORTICAL
OP2	BONE	SPONGIOUS / CORTICAL
OP3	BONE	SPONGIOUS / CORTICAL
OP3A	BONE	SPONGIOUS / CORTICAL
OP4	BONE	SPONGIOUS / CORTICAL
OP5	ROOT	PERIO
OP5A	ROOT	PERIO
	BONE	SPECIAL

INSERT	MODE	POWER
--------	------	-------

### OSTEOPLASTY

OP6	ROOT	PERIO
OP6A	ROOT	PERIO
OP7	BONE	SPONGIOUS / CORTICAL
OP8	ROOT	PERIO
	BONE	SPECIAL
OP9	BONE	SPONGIOUS / CORTICAL
SLC	BONE	SPONGIOUS / CORTICAL

### MINI IMPLANT PREP

MDI 1.9	IMPL	IMPLANT
MDI 2.2	IMPL	IMPLANT
MDI 2.5	IMPL	IMPLANT

### SINUS MEMBRANE ELEVATION

EL1	ROOT	ENDO*
EL2	ROOT	ENDO*
EL3	ROOT	ENDO*
SLS	BONE	SPECIAL*
SLE1	BONE	SPECIAL*
SLE2	BONE	SPECIAL*

### PERIOSTEUM

PR1	BONE	SPONGIOUS / CORTICAL
PR2	BONE	SPONGIOUS / CORTICAL

### ENDO RETRO

EN1	ROOT	ENDO*
EN2	ROOT	ENDO*
EN3	ROOT	ENDO*
EN4	ROOT	ENDO*
EN5R	ROOT	ENDO*
EN5L	ROOT	ENDO*
EN6R	ROOT	ENDO*
EN6L	ROOT	ENDO*

### EXTRACTION

EX1	BONE	SPONGIOUS / CORTICAL
EX2	BONE	SPONGIOUS / CORTICAL
EX3	BONE	SPONGIOUS / CORTICAL

### EXPLANTATION

EXP3-R	BONE	SPECIAL / SPONGIOUS / CORTICAL
EXP3-L	BONE	SPECIAL / SPONGIOUS / CORTICAL
EXP4-R	BONE	SPECIAL / SPONGIOUS / CORTICAL
EXP4-L	BONE	SPECIAL / SPONGIOUS / CORTICAL

### PERIO

PP1	ROOT	PERIO
PS1	ROOT	PERIO
PS2	ROOT	PERIO
PS6	ROOT	PERIO
PP10	ROOT	PERIO
PP11	ROOT	PERIO
PP12	ROOT	PERIO

### IMPLANT CLEANING

ICP + IC1	ROOT	ENDO / PERIO
-----------	------	--------------

### RESTORATIVE

DB2	BONE	SPECIAL
	ROOT	PERIO

\* Massima potenza consentita  
\* Maximum power permitted

\* Maximal zugelassene Leistung  
\* Puissance maximum autorisée

\* Máxima potencia permitida  
\* Maximalt tillåten effekt



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	MODE	POWER**
<b>IMPLANT PREP</b>		
IM1S	BONE	SPECIAL*
IM1 AL	BONE	1 / 2 / 3
IM1 SP	BONE	1 / 2 / 3
IM2A	BONE	1 / 2 / 3
IM2A-15	BONE	1 / 2 / 3
IM2P	BONE	1 / 2 / 3
IM2P-15	BONE	1 / 2 / 3
IM2.8A	BONE	1 / 2 / 3
IM2.8P	BONE	1 / 2 / 3
IM3A	BONE	1 / 2 / 3
IM3A -15	BONE	1 / 2 / 3
IM3P	BONE	1 / 2 / 3
IM3P-15	BONE	1 / 2 / 3
IM3.4A	BONE	1 / 2 / 3
IM3.4P	BONE	1 / 2 / 3
IM4A	BONE	1 / 2 / 3
IM4P	BONE	1 / 2 / 3
P2-3	BONE	1 / 2 / 3
P3-4	BONE	1 / 2 / 3

INSERT	MODE	POWER**
<b>OSTEOTOMY</b>		
OT1	BONE	1 / 2 / 3
OT1A	BONE	1 / 2 / 3
OT2	BONE	1 / 2 / 3
OT3	BONE	1 / 2 / 3
OT4	BONE	1 / 2 / 3
OT5	BONE	1 / 2 / 3
OT5A	BONE	1 / 2 / 3
OT5B	BONE	1 / 2 / 3
OT6	BONE	1 / 2 / 3
OT7	BONE	1 / 2 / 3
OT7A	BONE	1 / 2 / 3
OT7S-3	BONE	SPECIAL*
OT7S-4	BONE	SPECIAL*
OT7-20	BONE	1 / 2 / 3
OT8L	BONE	1 / 2 / 3
OT8R	BONE	1 / 2 / 3
OT9	BONE	1 / 2 / 3
OT11	BONE	1 / 2 / 3
OT12	BONE	1 / 2 / 3
OT12S	BONE	SPECIAL*
OT13	BONE	1 / 2 / 3
OT14	BONE	1 / 2 / 3
SLO-H	BONE	1 / 2 / 3
PL1	BONE	1 / 2 / 3
PL2	BONE	1 / 2 / 3
PL3	BONE	1 / 2 / 3

INSERT	MODE	POWER**
<b>OSTEOPLASTY</b>		
OP1	BONE	1 / 2 / 3
OP2	BONE	1 / 2 / 3
OP3	BONE	1 / 2 / 3
OP3A	BONE	1 / 2 / 3
OP4	BONE	1 / 2 / 3
OP5	ROOT	PERIO
OP5A	ROOT	PERIO
	BONE	SPECIAL

INSERT	MODE	POWER**
<b>OSTEOPLASTY</b>		
OP6	ROOT	PERIO
OP6A	ROOT	PERIO
OP7	BONE	1 / 2 / 3
OP9	BONE	1 / 2 / 3
SLC	BONE	1 / 2 / 3

INSERT	MODE	POWER**
<b>MINI IMPLANT PREP</b>		
MDI 1.9	BONE	1 / 2 / 3
MDI 2.2	BONE	1 / 2 / 3
MDI 2.5	BONE	1 / 2 / 3

INSERT	MODE	POWER**
<b>SINUS MEMBRANE ELEVATION</b>		
EL1	ROOT	ENDO*
EL2	ROOT	ENDO*
EL3	ROOT	ENDO*
SLS	BONE	SPECIAL*
SLE1	BONE	SPECIAL*
SLE2	BONE	SPECIAL*

INSERT	MODE	POWER**
<b>PERIOSTEUM</b>		
PR1	BONE	1 / 2 / 3
PR2	BONE	1 / 2 / 3

INSERT	MODE	POWER**
<b>ENDO RETRO</b>		
EN1	ROOT	ENDO*
EN2	ROOT	ENDO*
EN3	ROOT	ENDO*
EN4	ROOT	ENDO*
EN5R	ROOT	ENDO*
EN5L	ROOT	ENDO*
EN6R	ROOT	ENDO*
EN6L	ROOT	ENDO*

INSERT	MODE	POWER**
<b>EXTRACTION</b>		
EX1	BONE	1 / 2 / 3
EX2	BONE	1 / 2 / 3
EX3	BONE	1 / 2 / 3

INSERT	MODE	POWER**
<b>EXPLANTATION</b>		
EXP3-R	BONE	SPECIAL / 3 / 2 / 1
EXP3-L	BONE	SPECIAL / 3 / 2 / 1
EXP4-R	BONE	SPECIAL / 3 / 2 / 1
EXP4-L	BONE	SPECIAL / 3 / 2 / 1

INSERT	MODE	POWER**
<b>PERIO</b>		
PP1	ROOT	PERIO
PS1	ROOT	PERIO
PS2	ROOT	PERIO
PS6	ROOT	PERIO
PP10	ROOT	PERIO
PP11	ROOT	PERIO
PP12	ROOT	PERIO

INSERT	MODE	POWER**
<b>IMPLANT CLEANING</b>		
ICP + IC1	ROOT	ENDO / PERIO

INSERT	MODE	POWER**
<b>RESTORATIVE</b>		
DB2	BONE	SPECIAL
	ROOT	PERIO

\* Massima potenza consentita \* Maximal zugelassene Leistung  
 \* Maximum power permitted \* Puissance maximum autorisée

\* Máxima potencia permitida  
 \* Maximalt tillåten effekt

\*\* POWER: 1 = max 2 = med 3 = min



- IT** → Settaggi appropriati per gli inserti
- EN** → Appropriate settings for the inserts
- DE** → Korrekte Einstellungen für die Instrumente
- FR** → Réglages appropriés pour les inserts
- ES** → Ajustes apropiados para los insertos
- SV** → Korrekta Inställningar för spetsarna

INSERT	MODE	POWER
<b>IMPLANT PREP</b>		
IM1S	BOOSTED	C
IM1 AL	BOOSTED	C
IM1 SP	BOOSTED	C
IM2A	BOOSTED	C
IM2A-15	BOOSTED	C
IM2P	BOOSTED	C
IM2P-15	BOOSTED	C
IM2.8A	BOOSTED	C
IM2.8P	BOOSTED	C
IM3A	BOOSTED	C
IM3A -15	BOOSTED	C
IM3P	BOOSTED	C
IM3P-15	BOOSTED	C
IM3.4A	BOOSTED	C
IM3.4P	BOOSTED	C
IM4A	BOOSTED	C
IM4P	BOOSTED	C
P2-3	BOOSTED	C
P3-4	BOOSTED	C

INSERT	MODE	POWER
<b>OSTEOTOMY</b>		
OT1	BOOSTED	C
OT1A	BOOSTED	C
OT2	BOOSTED	C
OT3	BOOSTED	C
OT4	BOOSTED	C
OT5	BOOSTED	C
OT5A	BOOSTED	C
OT5B	BOOSTED	C
OT6	BOOSTED	C
OT7	BOOSTED	C
OT7A	BOOSTED	C
OT7S-3	BOOSTED	C
OT7S-4	BOOSTED	C
OT7-20	BOOSTED	C
OT8L	BOOSTED	C
OT8R	BOOSTED	C
OT9	BOOSTED	C
OT11	BOOSTED	C
OT12	BOOSTED	C
OT12S	BOOSTED	C
OT13	BOOSTED	C
OT14	BOOSTED	C
SLO-H	BOOSTED	C
PL1	BOOSTED	C
PL2	BOOSTED	C
PL3	BOOSTED	C

INSERT	MODE	POWER
<b>OSTEOPLASTY</b>		
OP1	BOOSTED	C
OP2	BOOSTED	C
OP3	BOOSTED	C
OP3A	BOOSTED	C
OP4	BOOSTED	C
OP5	HIGH	2 - 3
OP5A	HIGH	2 - 3

INSERT	MODE	POWER
<b>OSTEOPLASTY</b>		
OP6	HIGH	2 - 3
OP6A	HIGH	2 - 3
OP7	BOOSTED	C
OP9	BOOSTED	C
SLC	BOOSTED	C

INSERT	MODE	POWER
<b>MINI IMPLANT PREP</b>		
MDI 1.9	BOOSTED	C
MDI 2.2	BOOSTED	C
MDI 2.5	BOOSTED	C

INSERT	MODE	POWER
<b>SINUS MEMBRANE ELEVATION</b>		
EL1	LOW	1
EL2	LOW	1
EL3	LOW	1
SLS	BOOSTED	C
SLE1	BOOSTED	C
SLE2	BOOSTED	C

INSERT	MODE	POWER
<b>PERIOSTEUM</b>		
PR1	BOOSTED	C
PR2	BOOSTED	C

INSERT	MODE	POWER
<b>ENDO RETRO</b>		
EN1	LOW	1
EN2	LOW	1
EN3	LOW	1
EN4	LOW	1
EN5R	LOW	1
EN5L	LOW	1
EN6R	LOW	1
EN6L	LOW	1

INSERT	MODE	POWER
<b>EXTRACTION</b>		
EX1	BOOSTED	C
EX2	BOOSTED	C
EX3	BOOSTED	C

INSERT	MODE	POWER
<b>EXPLANTATION</b>		
EXP3-R	BOOSTED	C
EXP3-L	BOOSTED	C
EXP4-R	BOOSTED	C
EXP4-L	BOOSTED	C

INSERT	MODE	POWER
<b>PERIO</b>		
PP1	HIGH	2 - 3
PS1	HIGH	2 - 3
PS2	HIGH	2 - 3
PS6	HIGH	2 - 3
PP10	HIGH	2 - 3
PP11	HIGH	2 - 3
PP12	HIGH	2 - 3

INSERT	MODE	POWER
<b>IMPLANT CLEANING</b>		
ICP + IC1	HIGH	2 - 3
	LOW	1

INSERT	MODE	POWER
<b>RESTORATIVE</b>		
DB2	BOOSTED	A
	HIGH	2 - 3

\* Massima potenza consentita  
\* Maximum power permitted

\* Maximal zugelassene Leistung  
\* Puissance maximum autorisée

\* Máxima potencia permitida  
\* Maximalt tillåten effekt